Models 1500SC and 3000SC
Modelos 1500SC y 3000SC
Modèles 1500SC et 3000SC

Quick-Fit™ Reusable Stopcock Canister

**(For use with: Model 1504, 1500cc Quick-Fit Liner,
Model 3004, 3000cc Quick-Fit Liner)**

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Quick-Fit™ Reusable Stopcock Canister

(For use with: Model 1504, 1500cc Quick-Fit Liner,
Model 3004, 3000cc Quick-Fit Liner)

INTENDED USE: Intended for use with suction equipment to allow for the collection and disposal of bodily fluids or other materials removed during surgical or medical procedures.

DIRECTIONS FOR USE

(Single Canister/Liner Only)

Note: If more than one canister/liner is needed, a manifold must be used. Use manifold model number 538410.

1. Stopcock canister may be used on a single wall plate, regulator mounted, or on a four-canister stand.
2. If used on a non-Bemis single wall plate, place adapting Quick-Fit wire hanger (Model 539010) into wall plate and slide stopcock canister into it.
3. If the stopcock canister needs to be regulator mounted, utilize the adapting regulator bracket for Quick-Fit (Model 538510).

Note: Bemis does not recommend placing a 3000cc canister on a regulator.

4. Make sure stopcock is in OFF position. (Figure 3)
5. Attach vacuum tube from regulator or vacuum outlet to stopcock canister tubing barb underneath stopcock. (Figure 1)
6. Insert Quick-Fit liner into reusable canister. Press lid firmly around perimeter.
7. Connect white vacuum hose to vacuum port of Quick-Fit liner.
8. Attach patient tube to patient port and cap all other unused ports.
9. Turn vacuum on, then turn stopcock to ON position. Liner should now be inflated.

Note: Stopcock is ON when vertical (Figure 2) and OFF when horizontal (Figure 3).

10. Before disconnecting vacuum at end of procedure, remove patient port tube and cap all ports tightly. Turn off stopcock; remove white vacuum hose and cap vacuum port.
11. Carefully remove and dispose of liner according to hospital policy; clean and disinfect reusable outer canister immediately after use.

Note: When using a tandem setup, dispose of all used and unused liners after the procedure.

Figure 1

Tubing barb

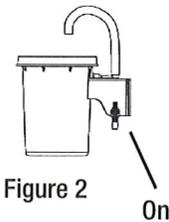
Figure 2

On

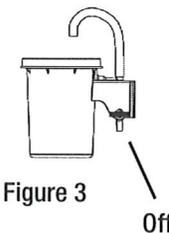
Figure 3

Off

⚠ CAUTION

- **ⓧ Liner is single use only. Outer canister is reusable.**
- **Always** use matching size liner and outer canister. **Do not** attempt to use a 1500cc liner in a 3000cc canister.
- Not recommended for use with portable vacuum systems as reduced airflow may result.
- Clean outer canister with 10% bleach or 10% alcohol solution. **Do not** use acetone-based cleaners.
- **Do not** exceed vacuum level of 25" (640mm) Hg.
- **Do not** apply continuous vacuum longer than 24 hours.
- Not intended as a measuring device, only for general reference, not specific measurement.
-  Check expiration date on top of liner lid for use-by date.
-  Store liner in a dark place. Long term exposure to light may compromise product performance and result in breakage during use.

Report any product malfunctions or complaints to Bemis Manufacturing Co. using the contact information on back page.

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