# Wall Safe<sup>®</sup> Bracket Mounting Instructions

Model 410 020 / Modelo 410 020 / Modèle 410 020

## TOP / PARTE SUPERIOR / DESSUS

Use Wall Safe Bracket Key #415 to remove sharps container from bracket.

## MASONRY WALLS

1. Using the Wall Safe Bracket as a template, locate it on the wall in the desired position and drill locating reference points at each screw hole.
2. Drill 1/4" diameter holes 1-1/4" deep. Blow dust out of the holes and insert plastic wall plugs, tapping them with a hammer if needed to bring them flush to the wall. Screw on the Wall Safe Bracket with the provided screws.

**Note:** If for any reason the provided screws are not used, only pan head or round head 10-gauge screws should be substituted. In no case should countersunk screws be used.

## INCLUDED IN PACKAGE

- 10 Bemis Wall Safe Brackets
- 44 Plastic Wall Plugs
- 44 1-1/4" Pan Head 10-Gauge Screws

Mount Wall Safe Bracket at a safely accessible height approximately 50" from top edge to floor.

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## HOLLOW WALLS

Metal expanding wall fasteners that require a 3/8" drill hole size or plastic toggle expanding wall fasteners that require a 5/16" drill hole size should be used. Fasteners requiring smaller diameter holes have screw heads that are too small for secure wall bracket fastening.

## Typical recommended fasteners are:

Molly fastener	CD6S	Metal fastener for 3/8" to 5/8" wallboard
Rawl anchor	2112	Metal fastener for 3/8" to 5/8" wallboard
Poly-Toggle anchor	2304	Plastic toggle fastener for 1/2" wallboard
Poly-Toggle anchor	2305	Plastic toggle fastener for 5/8" wallboard

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