

Directions for Use Manifold with Tandem Tubing


Model#	Description
538410	4-Liner Stopcock Manifold
536510	18" Tandem Tubing
534910	4-Canister Stand

1. Clip manifold to top of stand.
2. Attach tubing connections of manifold to the ports underneath the reusable canister/bracket.
3. Attach patient tube to the patient port of any of the four Quick-Fit liners.
4. Attach tandem tube to the tandem port of the first canister and to the patient port of the next canister.
5. Repeat the tandem-to-patient, tandem-to-patient connection until the last canister is reached.
6. Cap last tandem port and all pour spouts.
7. Connect vacuum tubing to the manifold vacuum port.
8. Open stopcock on manifold to all Quick-Fit liners.

When canisters are full:

1. LEAVE VACUUM ON AND STOPCOCKS OPEN while removing patient and tandem tubing.
2. Cap all pour spouts, patient ports and tandem ports.
3. Turn off all stopcocks.
4. Disconnect vacuum tube from vacuum port of all liners and cap ports.
5. Dispose of liners according to hospital policy.

Note: All used and unused liners must be discarded.

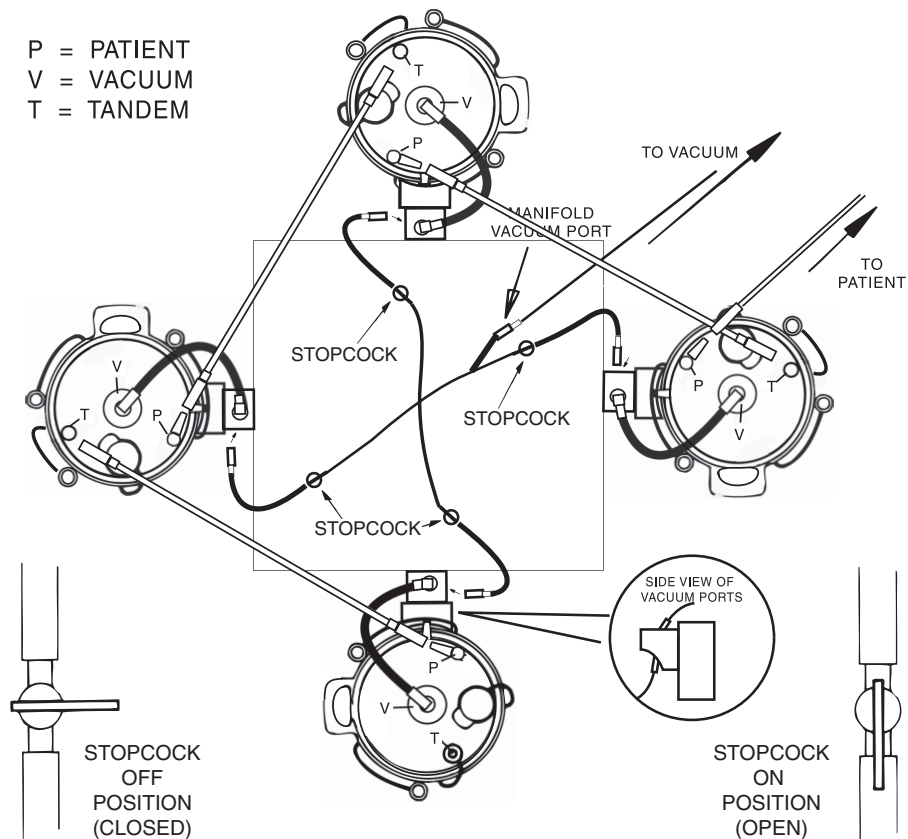

Directions for Use Manifold Only

Model#	
538410	4-Liner Stopcock Manifold
534910	4-Canister Stand

1. Clip manifold to top of stand.
2. Attach tubing connections of manifold to the ports underneath the reusable canister/bracket.
3. Attach patient tube to the patient port of any of the four Quick-Fit liners.
4. Cap pour spouts and tandem ports of all canisters. (The patient ports of the other canisters do not have to be capped if the stopcock for that canister is closed.)
5. Open stopcock for the first canister; close stopcocks for the other canisters.
6. Connect vacuum tubing to the manifold vacuum port.

When first canister is full:

1. Remove the patient tube and cap the patient port.
2. Close the stopcock.
3. Attach the patient tube to the patient port of the next canister.
4. Open stopcock for the second canister.
5. Disconnect vacuum tube from vacuum port of full canister.
6. Replace full liner if necessary and dispose of according to hospital policy.

Note: All used and unused liners must be discarded after procedure.

IMPORTANT: Be sure that the patient port is capped before closing stopcocks or disconnecting vacuum tubing.

