

Suction Canister System

2 Canister Stopcock Manifold

Clean, Ready to Use

CAUTION: Federal law (U.S.A.) restricts this device to sale by or on the order of a physician.

INSTRUCTIONS FOR USE

1. Attach tubing connectors of manifold to the vacuum port of canisters.
2. Attach patient tube to the patient port of either canister.
3. Cap the tandem port and pour spout of both canisters.
4. Attach vacuum source to the Y-connector.
5. Open stopcock to first canister only.
6. When full, move patient tubing to second canister and open stopcock to second canister.

To replace full canister, close stopcock, disconnect manifold and cap all ports before moving. Replace with new canister, set up as before.

