Wall Safe® Sharps Disposal Container

Models 125 and 150 / Modelos 125 y 150 / Modèles 125 et 150

ATTENTION

*Please review these instructions carefully
before using this container.*

- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only (ⓧ)
- Non-toxic, latex-free and heavy metal free (LATEX)

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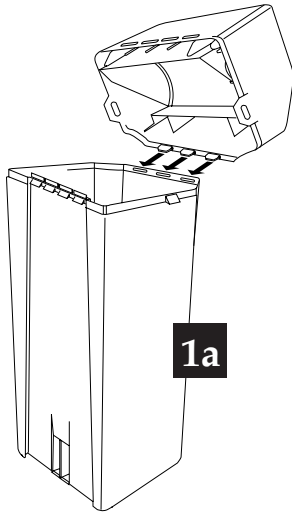
1a

1. To assemble

- Insert three tabs on cover into the three front slots in base.
- Rotate cover over base to snap tabs in slots on back and sides of container. Push down to lock cover in place.
- Check all tabs for a positive lock.

2. To mount in bracket

NOTE: This container must be wall-mounted during use.

Mount on wall using Wall Safe mounting bracket #410 020. Gently slide Wall Safe down into bracket until it clicks.

3. To use

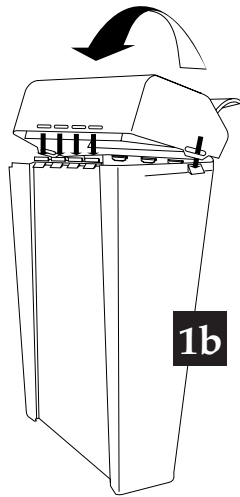
- Brush back lid with one-handed motion, dropping syringe horizontally
- Check fill level regularly. Replace container when fill level is reached.

4. Final closure

- When full, insert center tab into slot on lid for final closure.
- To remove Wall Safe from bracket, insert the bracket key, #415 020, into slot at bottom of unit, push up and release from bracket.

5. Disposal

Dispose of container according to facility policy. Wall Safes may be autoclaved or incinerated.

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