SharpSentinel® Wall Cabinet System

Installation Instructions / Directions for Use – Instrucciones de instalación / Instrucciones de uso – Instructions d'installation / Mode d'emploi

#1031 3-Gallon with #1041 Glove Box for use with #333 Sharps Containers

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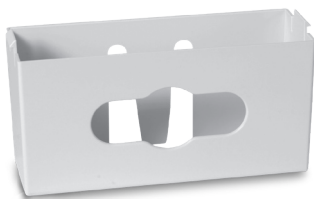
#1051 5-Quart with #1041 Glove Box for use with #175 Sharps Containers

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Read all precautions and instructions before installing and using the SharpSentinel® Wall Cabinet System. Improper installation or use may result in exposure to infectious materials and the potential of needlestick injuries.

PRECAUTIONS

- Never place used sharps in empty cabinet
- Cabinet should always be locked except when installing or removing sharps container.
- Use only with designated Bemis sharps containers.
- Always close and permanently lock lid before removal of sharps container.
- **IMPORTANT!** Do not allow sharps container to exceed recommended fill line.

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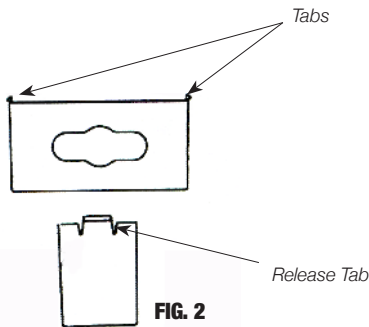
FIG. 2

SIDE VIEW

CLEANING INSTRUCTIONS FOR CABINET

Clean cabinet regularly to maintain appearance and function. Wipe exterior of cabinet with damp cloth and mild soap or disinfectant. Never clean with solvents such as acetone or alcohol. Do not attempt to clean interior of cabinet when sharps container is installed. Clean interior of cabinet using mild soap or disinfectant.

Installation Instructions

Cabinets can be installed using holes from existing units or on any new wall surface. Install at a recommended height of 53"-56" from top of cabinet to floor.

Note: Hardware not included. Use appropriate fasteners for your wall construction.

TO INSTALL ON NEW WALL SURFACE (FIGURE 1):

Do not drill holes without first checking for plumbing or wiring behind wall. Using back of cabinet as a template, mark hole locations with a pencil. Drilling at least four holes is recommended using mounting slots A, B, C or D. Drill holes and secure cabinet to wall using appropriate fasteners.

TO INSTALL USING EXISTING HOLES:

Loosen top fasteners left from existing bracket or cabinet and remove bottom fasteners. Slide new cabinet into position on top fasteners using keyhole slots. Replace bottom fasteners using any mounting slots C, D or below.

TO INSTALL OPTIONAL #1041 GLOVE BOX HOLDER TO CABINET (FIGURE 2):

Insert tabs on left and right side of glove box into matching slots on bottom of cabinet until it snaps into position.

TO REMOVE GLOVE BOX FROM CABINET:

Holding glove box with both hands, push in release tab on either left or right side while pulling down until glove box is removed.

Directions for use

TO INSERT SHARPS CONTAINER IN CABINET:

1. With biohazard label facing front, slide container straight into cabinet.
2. Close and lock cabinet door.

TO REMOVE FULL SHARPS CONTAINER FROM CABINET:

1. Insert final disposal tabs into slots on lid of sharps container. Push down and snap into place.
2. Unlock and open cabinet door.
3. Place both hands underneath handles and slide container out of cabinet. Dispose of according to hospital policy.
4. Insert new sharps container into cabinet; close and lock door.

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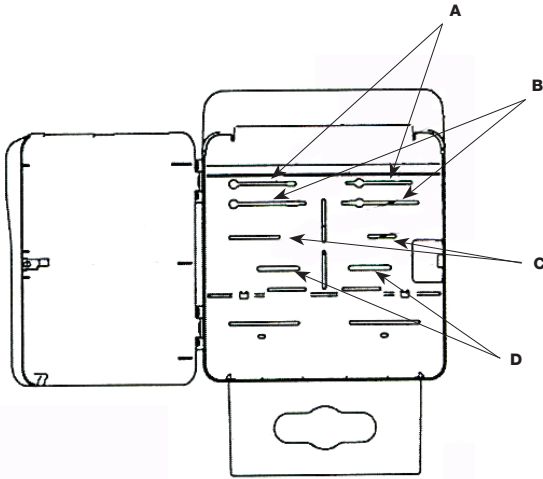
FIG. 1

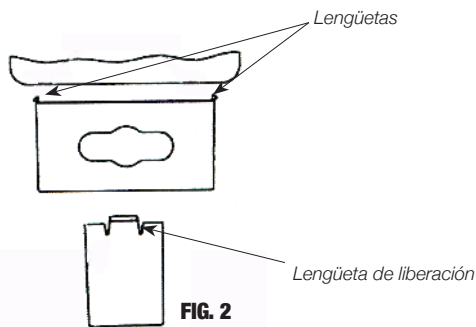
FIG. 2

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