# 8- and 11 Gallon Sharps and Chemotherapy Containers

Models 108, 208, 111 and 211 / Modelos 108, 208, 111 y 211 / Modèles 108, 208, 111 et 211 (30 L / 42 L)

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## ATTENTION

**Please review these instructions carefully  
before using this container.**

- For use in supervised and controlled areas
- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only. (X)
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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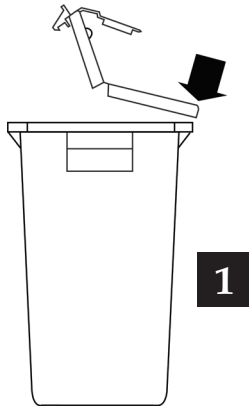
## 1. To assemble

- Place base on stable surface with biohazard label facing forward. Place lid on base with embossed writing toward front.
- Firmly push the back half of the lid onto the base until it snaps under the rim.
- Leave front half open to dispose of large items.

## 2. To use

Do not force sharps into container. Regularly check content to ensure it does not exceed the fill line.

## 3. For temporary closure

Gently push down on the front half of the lid, until front hooks engage rim.

**Note:** The round opening does not have a temporary close feature. Do not close and attempt to reopen by pulling the tab.

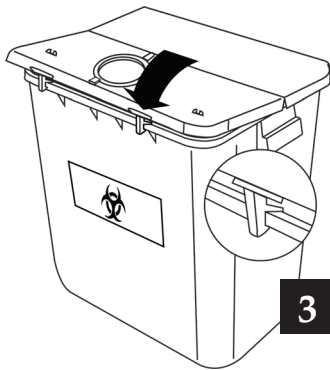
## 4. For final closure

- Lift both tabs at the front edge so that they clear the rim (*see A*).
- Firmly push on all the edges to ensure that the lid is snapped to the base all the way around.
- To close the small opening for final disposal, insert tab into slot and press down firmly (*see B*).

## 5. Disposal

Chemotherapy containers must be incinerated for final disposal.

Sharps containers may be autoclaved and/or incinerated for final disposal, according to facility policy.

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