

BEMIS QUICK-CONNECT II BRACKET CLEANING INSTRUCTIONS



RECOMMENDED CLEANING AGENT COMPOUND

Use Quaternary Ammonium Chloride cleaning agent

1. Follow the Quaternary Ammonium Chloride manufacturer's use instructions to wipe all external surfaces of Quick-Connect II.
2. Quick-Connect II does not need to be removed from the vacuum regulator for cleaning.
3. Quaternary Ammonium Chloride can leave a buildup of residue, it is recommended to wipe off residue with the use of a clean, dry or water dampened cloth or paper towel.

IMPORTANT: The use of non-recommended cleaning agents can cause corrosion and reduce the life of Quick-Connect II bracket.

- a. DO NOT use cleaning agents with Sodium Hypochlorite (bleach).
- b. DO NOT use cleaning agents with Hydrogen Peroxide.

