Hazardous Waste Pharmaceutical Container with Gasketed Lids and Absorbent Pads

2-Gallon Hazardous Waste Pharmaceutical Container / Contenedor de 2 galones para residuos farmacéuticos peligrosos /
Contenants à déchets pharmaceutiques dangereux de 7,6 litres (2 gallons) — Model 5002 / Modelo 5002 / Modèle 5002

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ATTENTION

**Please review these instructions carefully
before using this container.**

- For use in supervised and controlled areas.
- For solid hazardous pharmaceutical waste including P- and U-listed drugs.
- Not for sharps, infectious/biohazard waste and free liquids.
- For incineration only by federally permitted hazardous waste incinerators.
- Do not overfill or force items into container.
- If container is dropped, inspect for damage before handling.
- For single use only. ⓧ Maximum Usage: 90 days.
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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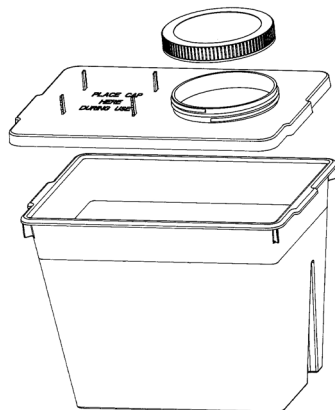
1. To assemble

- Check that container is not damaged before placing into use.
- Check that foam gasket in cover is aligned with positioning tabs.
- Set lid on base, lining up handles on both. Press lid firmly to base ensuring that three snaps in front and back and two snaps on each side are firmly engaged.

2. To mount in bracket

Mount on wall using wall mounting bracket #435 020. Gently slide container into bracket until it clicks.

3. To use

- With one hand, drop items into container. During use, the cap may be stored on the cap holder.
- Keep container closed when not in use.
- Check fill level regularly. Replace container when fill level is reached.

4. Final closure

- Screw lid on tightly for final disposal.
- To remove container from bracket, insert the bracket key, #440 020, into slots at bottom of unit, push up and release container from bracket.

5. Disposal

- Container can be in use for maximum of 90 days.
- Incineration only by a federally permitted hazardous waste incinerator.

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ATTENTION

Please review these instructions carefully before using this container.

- For use in supervised and controlled areas.
- For solid hazardous pharmaceutical waste including P- and U-listed drugs.
- Not for sharps, infectious/biohazard waste and free liquids.
- For incineration only by federally permitted hazardous waste incinerators.
- Do not overfill or force items into container.
- If container is dropped, inspect for damage before handling.
- For single use only. (X) Maximum Usage: 90 days.
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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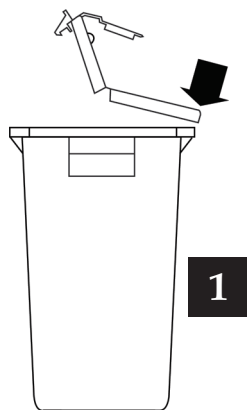
1. To assemble

- a. Check that container is not damaged before placing into use.
- b. Place base on stable surface with label facing forward. Place lid on base with round opening aligned with label.
- c. Firmly push back half of the lid onto the base until it snaps under the rim.
- d. Leave front half open for disposal of items.

2. To use

Do not force items into container. Regularly check content for fill level.

3. For temporary closure

Gently push down on the front half of the lid, until front hooks engage rim.

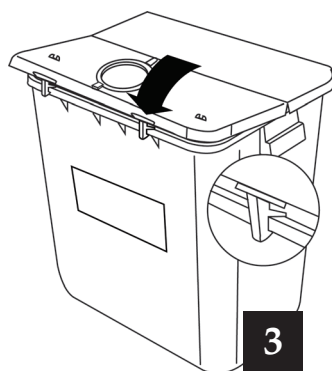
Note: The round opening does not have a temporary close feature. Do not close and attempt to reopen by pulling the tab.

4. For final closure

- a. Lift both tabs at the front edge so that they clear the rim (see A).
- b. Firmly push on all the edges to ensure that the lid is snapped to the base all the way around.
- c. To close the small opening for final disposal, insert tab into slot and press down firmly (see B).

5. Disposal

- a. Container can be in use for maximum of 90 days.
- b. Incineration only by a federally permitted hazardous waste incinerator.

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