Hazardous Waste Pharmaceutical Container with Gasketed Lids and Absorbent Pads

8 and 11-Gallon Hazardous Waste Pharmaceutical Containers / Contenedores de 8 y 11 galones para residuos farmacéuticos peligrosos / Contenants à déchets pharmaceutiques dangereux de 30 et 42 litres (8 et 11 gallons)

Models 5008 and 5011 / Modelos 5008 y 5011 / Modèles 5008 et 5011

ATTENTION

Please review these instructions carefully before using this container.

- For use in supervised and controlled areas.
- For solid hazardous pharmaceutical waste including P- and U-listed drugs.
- Not for sharps, infectious/biohazard waste and free liquids.
- For incineration only by federally permitted hazardous waste incinerators.
- Do not overfill or force items into container.
- If container is dropped, inspect for damage before handling.
- For single use only. (X) Maximum Usage: 90 days.
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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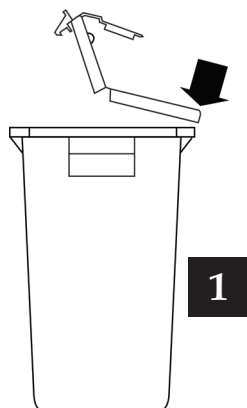
1. To assemble

- Check that container is not damaged before placing into use.
- Place base on stable surface with label facing forward. Place lid on base with round opening aligned with label.
- Firmly push back half of the lid onto the base until it snaps under the rim.
- Leave front half open for disposal of items.

2. To use

Do not force items into container. Regularly check content for fill level.

3. For temporary closure

Gently push down on the front half of the lid, until front hooks engage rim.

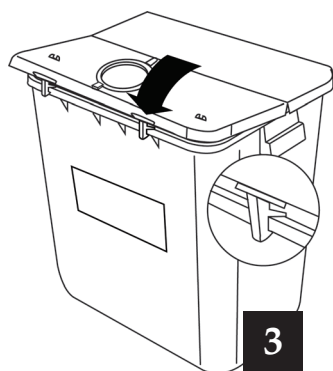
Note: The round opening does not have a temporary close feature. Do not close and attempt to reopen by pulling the tab.

4. For final closure

- Lift both tabs at the front edge so that they clear the rim (see A).
- Firmly push on all the edges to ensure that the lid is snapped to the base all the way around.
- To close the small opening for final disposal, insert tab into slot and press down firmly (see B).

5. Disposal

- Container can be in use for maximum of 90 days.
- Incineration only by a federally permitted hazardous waste incinerator.

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