2-Gallon Chemotherapy Container with Gasketed Lid and Screw-On Cap

Model 202 / Modelo 202 / Modèle 202 (7,6 L)

ATTENTION

Please review these instructions carefully before using this container.

- For use in supervised and controlled areas
- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only. (X)
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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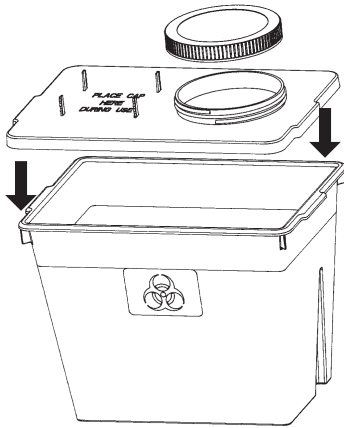
1. To assemble

- Set lid on base, lining up handles on both.
- Press lid firmly to base ensuring that three snaps in front and back, and two snaps on each side are firmly engaged.
- The parts will not snap together if not correctly aligned. Be sure handles on both lid and base meet.

2. To mount in bracket

- Mount on wall using wall mounting bracket #445 020. Gently slide container into bracket until it clicks.

3. To use

- With one hand, drop sharps into container. During use, the cap may be stored on the cap holder.
- Keep container closed when not in use.
- Check fill level regularly. Replace container when fill level is reached.

4. Final closure

- Screw lid on tightly for final disposal.
- To remove container from bracket, insert the bracket key into slots at bottom of unit, push up and release container from bracket.

5. Disposal

Chemotherapy container must be incinerated for final disposal.

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