**Quick-Fit™ Suction Liner System
with anti-reflux valve
(1000cc, 1500cc, and 3000cc)**

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8/2021 Rev. 1.0
©2021 5995001

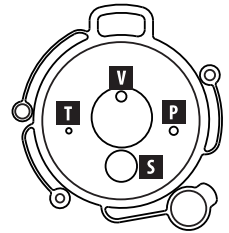
Quick-Fit™ Suction Liner System

INTENDED USE: Intended for use with suction equipment to allow for the collection and disposal of bodily fluids or other materials removed during surgical or medical procedures.

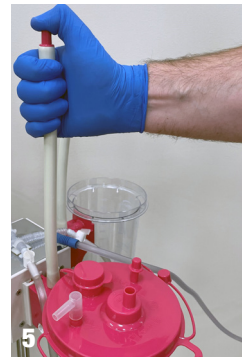
DIRECTIONS FOR USE

Note: If more than one canister/liner is needed, a manifold must be used. Use either manifold number 538210 (two-liner stopcock manifold) or 538410 (four-liner stopcock manifold).

1. Attach tube from vacuum source to port underneath bracket.
2. Gently shake lid to verify shut-off mechanism moves freely.
3. Grasp, fully stretch out flexible liner and add half twist of bag for easier insertion of liner into canister.
4. Insert liner into reusable outer canister and press lid firmly around entire perimeter to ensure proper seal.
5. To seat cover and inflate liner in canister, turn vacuum on to canister and place thumb over open end of white tube until liner is fully inflated.
6. Attach white vacuum hose to vacuum port. Connect patient tube to patient port or reference Manifold IFU for hooking in tandem. Cap any unused ports. **Liner should now be inflated and ready to use.**
7. At the end of the procedure, **and before disconnecting vacuum**, remove tubing and cap all ports lightly.
8. Grasp the handle to remove the liner from the outer canister and dispose of according to hospital policy. Clean and disinfect reusable outer canister after use.

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CAUTION

-  **Liner is single use only. Outer canister is reusable.**
- **Always** use matching color liner and outer canister.
- Liner contents are considered potentially hazardous, use appropriate PPE and handle accordingly.
- **Do not** apply continuous vacuum longer than 24 hours.
- **Do not** exceed vacuum level of 25" (640mm) Hg.
-  Check expiration date on top of liner lid for use-by date.
-  Store in dark place. Long term exposure to light may compromise product performance and result in breakage during use.
- Clean outer canister with 10% bleach or 10% alcohol solution after each use. **Do not** use acetone-based cleaners.
- Not intended as a measuring device, only for general reference, not specific measurement.

Loss of Vacuum: Check that vacuum is on, canister is properly sealed and that all connections are tight and tubing is not kinked. If loss continues, replace liner.

Report any product malfunctions or complaints to Bemis Manufacturing Co. using the contact information on back page.

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Bemis de Mexico S.A. de C.V.