

## Directions for Draining Canisters



	<p>Align the <b>Handle</b> on the Hi-Flow Canister in between the alignment <b>Pins</b>.</p>
	<p>Remove the <b>Tandem Port Cap</b> as well as the large <b>Pour Spout Cap</b> on the Hi-Flow Canister cover.</p> <p><b>IMPORTANT:</b> Be sure the caps are firmly affixed to the <b>Patient Port</b> and <b>Vacuum Port</b>.</p>
	<p>Rotate the <b>Drainhead</b> in the counterclockwise direction onto the top of the Hi-Flow Canister to seal drainhead to the Tandem and Pour Ports. The Drainhead will lock once it has been lowered into the correct position.</p>
	<p>Grasp the <b>Handle</b> and rotate the swingarm in the clockwise direction until the swingarm comes in contact with the stop. The canister will automatically drain.</p>
	<p>If you would like to rinse the interior of the canister, press the <b>Rinse Button</b> at this time.</p>
	<p>After the canister has drained, grasp the <b>Handle</b> and rotate the swingarm in the counterclockwise direction until the swingarm is in the home position.</p>
	<p>Grasp the <b>Release Lever</b> on the bottom of the drainhead and rotate the <b>Drainhead</b> clockwise until it rests against its stop. Recap all ports and dispose of according to hospital policy.</p>

## Directions for Draining Large Volume Canisters

	<p>Attach <b>Disposable Drain Tube</b> to the <b>Permanent Drain Hose</b> and place tube in <b>Canister</b>.</p>
	<p>Rotate <b>Handle</b> up to start draining canister.</p>
	<p>Once canister is drained:</p> <ul style="list-style-type: none"> <li>• rotate <b>Handle</b> down to off position</li> <li>• detach <b>Tube</b> and leave in canister for disposal</li> <li>• hang <b>Hose</b> on unit</li> </ul>

**Daily Maintenance** - Run 3000cc of water/PH neutral enzymatic cleaner mixture through the unit at the end of the day.

**For Service Contact:**

Name: \_\_\_\_\_

Phone: \_\_\_\_\_