

Directions for Draining Liners



	<p>Align the Handle on the Quick-Fit™ Liner in between the alignment Pins. Press the liner firmly onto the sealing edge of the canister.</p>
	<p>Remove the Patient Port Cap as well as the large Pour Spout Cap on the Quick-Fit™ Liner cover.</p> <p>IMPORTANT: Be sure the caps are firmly affixed to the Tandem Port and Vacuum Port.</p>
	<p>Rotate the Drainhead in the counterclockwise direction onto the top of the Quick-Fit™ Liner to seal drainhead to the patient and pour ports. The drainhead will lock once it has been lowered into the correct position.</p>
	<p>Grasp the Handle and rotate the swingarm in the clockwise direction until the swingarm comes in contact with the stop. The liner will automatically drain.</p>
	<p>If you would like to rinse the interior of the liner, press the Rinse Button at this time.</p>
	<p>After the liner has drained, grasp the Handle and rotate the swingarm in the counterclockwise direction until the swingarm is in the home position.</p>
	<p>Grasp the Release Lever on the bottom of the drainhead and rotate the Drainhead clockwise until it rests against its stop. Recap all ports and dispose of according to hospital policy.</p>

Directions for Draining Large Volume Canisters

	<p>Attach Disposable Drain Tube to the Permanent Drain Hose and place tube in Canister.</p>
	<p>Rotate Handle up to start draining canister.</p>
	<p>Once canister is drained:</p> <ul style="list-style-type: none"> • rotate Handle down to off position • detach Tube and leave in canister for disposal • hang Hose on unit

Daily Maintenance - Run 3000cc of water/PH neutral enzymatic cleaner mixture through the unit at the end of the day.

For Service Contact:

Name: _____

Phone: _____