


DIRECTIONS FOR USE (Single Canister/Liner Only)

For use with:

1004R 1000cc Suction Liner

**Note: If more than one canister/liner is needed, a manifold must be used.
Use manifold model number 538410.**

1. Stopcock canister may be used on a single wall plate, regulator mounted or on a four-canister stand.
2. If used on a non-Bemis single wall plate, place adapting wire hanger (Model 539010) into wall plate and slide stopcock canister into it.
3. If the stopcock canister needs to be regulator mounted, utilize the adapting regulator bracket (Model 538510). Note: Bemis does not recommend placing a 3000cc canister on a regulator.
4. Make sure stopcock is in OFF position. (Figure 3)
5. Attach vacuum tube from regulator or vacuum outlet to stopcock canister tubing barb underneath stopcock. (Figure 1)
6. Insert liner into reusable canister. Press lid firmly around perimeter.
7. Connect white vacuum hose to vacuum port of liner.
8. Attach patient tube to patient port and cap all other unused ports.
9. Turn vacuum on, then turn stopcock to ON position. Liner should now be inflated. Note: Stopcock is ON when vertical (Figure 2) and OFF when horizontal (Figure 3).
10. Before disconnecting vacuum at end of procedure, remove patient port tube and cap all ports tightly. Turn off stopcock; remove white vacuum hose and cap vacuum port.
11. Carefully remove and dispose of liner according to hospital policy; clean and disinfect reusable outer canister immediately after use.

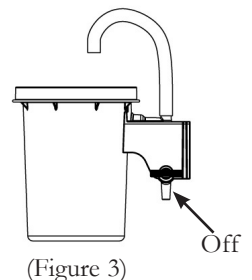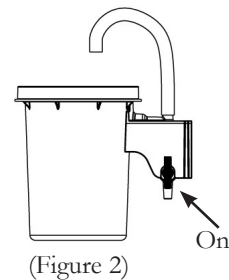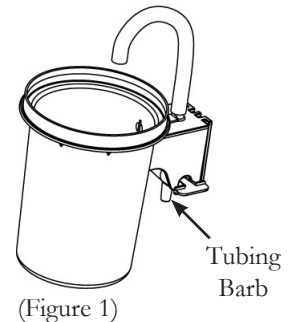Note: When using a tandem setup, dispose of all used and unused liners after the procedure.

CAUTION

- Liner is single use only. Outer canister is reusable.
- Always use matching size liner and outer canister. Do not attempt to use a 1000cc liner in a 3000cc canister.
- Not recommended for use with portable vacuum systems as reduced airflow may result.
- Clean outer canister with 10% bleach or 10% alcohol solution. Do not use acetone-based cleaners.
- Do not exceed vacuum level of 25" (640mm) Hg.
- Do not apply continuous vacuum longer than 24 hours.
- Not intended as a measuring device, only for general reference, not specific measurement.
- Store in a dark place. Long term exposure to light may compromise product performance and result in breakage during use.

Check expiration date on top of liner lid for use-by date.

Report any product malfunctions or complaints to Bemis Manufacturing Co. using the contact information below.

See brochure for complete line of canisters, hardware and accessories.

Distributed By: Bemis de México, Av. Texas 200, Parque Industrial Nacional, Ciénega de Flores, N.L. 65550, Mexico • 81-81545500 • FAX 81-83197884

Bemis Chile, Nueva Costanera 4229, Oficina 205, Santiago Chile • 800-2-23647 • FAX 56-2-2064697

Manufactured by: Bemis Manufacturing Company • 300 Mill Street, Sheboygan Falls, WI 53085 • 920-467-4621 • 800-558-7651 • FAX 920-467-8573

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