**Model 538410**  
**Modelo 538410**  
**Modèle 538410**

---

**Quick-Fit™ Suction Liner System**  
**4-Canister Stopcock Manifold**

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# Directions for Use

## MANIFOLD WITH TANDEM TUBING

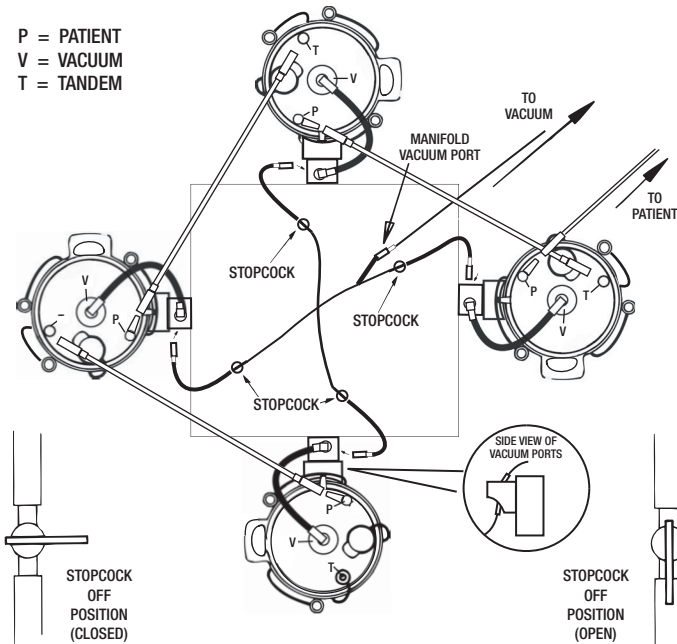
1. Clip manifold to top of stand.
2. Attach tubing connections of manifold to the ports underneath the reusable canister/bracket.
3. Attach patient tube to the patient port of any of the four Quick-Fit liners.
4. Attach tandem tube to the tandem port of the first canister and to the patient port of the next canister.
5. Repeat the tandem-to-patient, tandem-to-patient connection until the last canister is reached.
6. Cap last tandem port and all pour spouts.
7. Connect vacuum tubing to the manifold vacuum port.
8. Open stopcock on manifold to all Quick-Fit liners.

MODEL #	
538410	4-Liner Stopcock Manifold
537010	18" Tandem Tubing (25/case)
536510	18" Tandem Tubing (100/case)
534910	4-Canister Stand

## WHEN CANISTERS ARE FULL:

1. LEAVE VACUUM ON AND STOPCOCKS OPEN while removing patient and tandem tubing.
2. Cap all pour spots, patient ports, and tandem ports.
3. Turn off all stopcocks.
4. Disconnect vacuum tube from vacuum port of all liners and cap ports.
5. Dispose of liners according to hospital policy.

**Note:** All used and unused liners must be discarded.

# Quick-Fit™ Suction Liner System 4-Canister Stopcock Manifold

## MANIFOLD ONLY

1. Clip manifold to top of stand.
2. Attach tubing connections of manifold to the ports underneath the reusable canister/bracket.
3. Attach patient tube to the patient port of any of the four Quick-Fit liners.
4. Cap pour spouts and tandem ports of all canisters. (The patient ports of the other canisters do not have to be capped if the stopcock for that canister is closed.)
5. Open stopcock for the first container; close stopcocks for the other containers.
6. Connect vacuum tubing to the manifold vacuum port.

MODEL #

538410

4-Liner Stopcock Manifold

534910

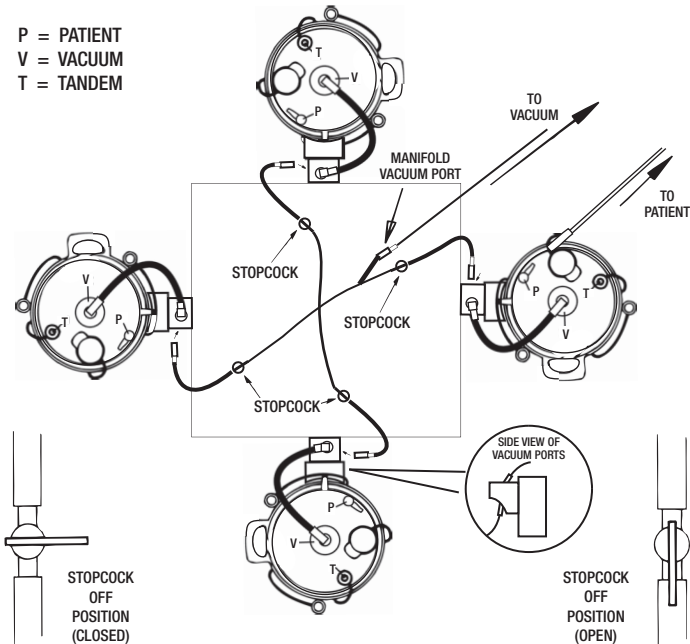
4-Canister Stand

## WHEN CANISTERS ARE FULL:

1. Remove the patient tube and cap the patient port.
2. Close the stopcock.
3. Attach the patient tube to the patient port of the next canister.
4. Open stopcock for the second canister.
5. Disconnect vacuum tube from the vacuum port of full canister.
6. Replace full liner if necessary and dispose of according to hospital policy.

**Note:** All used and unused liners must be discarded after procedure.

**IMPORTANT:** Be sure that the patient port is capped before closing stopcocks or disconnecting vacuum tubing.

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©2020 5995080 005