# SPECIMEN ADAPTOR

<b>SPECIMEN ADAPTOR</b>	<b>2</b>
<b>ADAPTADOR DE MUESTRAS</b>	<b>3</b>
<b>ADAPTATEUR D'ÉCHANTILLON</b>	<b>4</b>
<b>PROBENADAPTER</b>	<b>5</b>
<b>ADATTATORE PER CAMPIONI</b>	<b>6</b>
<b>ADAPTADOR DE AMOSTRAS</b>	<b>7</b>

Conforms to ISO10079-3:2014  
Cumplen con ISO10079-3:2014  
Conforme à ISO10079-3:2014  
Erfüllt die Anforderungen nach ISO10079-3:2014  
Conforme ai requisiti ISO10079-3:2014  
Em conformidade com a ISO 10079-3:2014

# SPECIMEN ADAPTOR


**INTENDED USE:** Intended for connection to suction canisters to allow for the collection of specimens removed during surgical or medical procedures.

## DIRECTIONS FOR USE - ASSEMBLY

**Note:** Begin directions only after properly assembling suction canister.


1. Insert Sock into Pour Spout.
2. Inspect that Specimen Adaptor is sealed to Canister Cover.
3. Inspect all connections.
4. Turn on vacuum source and test for any leakage.
5. Collect desired sample.

### SPECIMEN ADAPTOR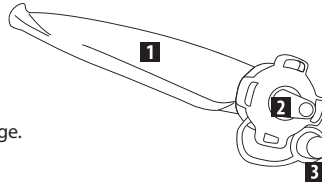


- 1 SOCK
- 2 PATIENT PORT
- 3 CAP


### CANISTER COVER

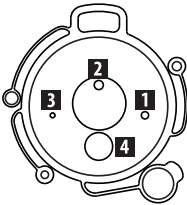SUCTION LINER

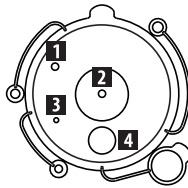HI-FLOW

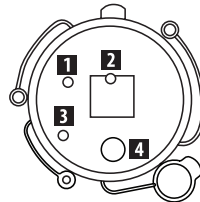HYDROPHOBIC

- 1 PATIENT
- 2 VACUUM
- 3 TANDEM
- 4 POUR SPOUT

## REMOVING SAMPLE

1. Remove tubing from Specimen Adaptor and cap port.
2. Remove Specimen Adaptor using caution.

## DISPOSAL

- Dispose Specimen Adaptor according to your facility's standard operation procedures for biohazard disposal and local disposal laws.

## TROUBLESHOOTING

- Loss of Vacuum: Check that vacuum is on, canister is properly sealed, and that all connections are tight and tubing is not kinked. If loss continues, replace canister.
- Report any product malfunctions or complaints to Bemis Health Care at [Customer.Service@BemisHealthCare.com](mailto:Customer.Service@BemisHealthCare.com)

## ⚠ CAUTION

- **ⓧ Single Use Only. Do not attempt to clean, sterilize or reuse specimen adaptor.** Possible consequence of reuse include exposure to pathogens.
- **Do not** exceed vacuum level of 25" (640mm) Hg.

- Contents are considered potentially hazardous. Use appropriate PPE and handle accordingly.
- ☀ Store in a dark place. Long term exposure to light may compromise product performance and result in breakage during use.

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