# 3-Gallon Sharps Container with Rotor Lid

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## **ATTENTION**

**Please review these instructions carefully  
before using this container.**

- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only. (X)
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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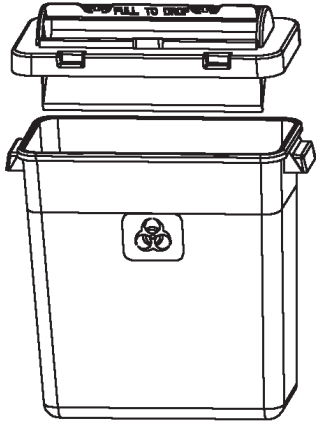
## 1. To assemble

- Align front of lid (lid opening) to front of base (biohazard label).
- Align tabs to holes in base. If they do not align, make sure lid and base are both facing forward.
- Press lid to base until all tabs snap into slots and are secure.

## 2. To mount in bracket

**NOTE:** This container must be wall-mounted during use.

- Mount on wall using wall mounting bracket #423 000 (non-locking) or #425 000 (locking). Gently slide container down into bracket.
- Can also be used with #1031 locking wall cabinet.

## 3. To use

- Place sharps horizontally into cradle opening. Pull cylinder forward if necessary to drop sharps into container.
- Check fill level regularly. Replace container when fill level is reached.

## 4. Final closure

- When full, insert tabs on cylinder closure into slots and press firmly.
- Remove container from bracket.

## 5. Disposal

Dispose of container according to facility policy. May be autoclaved or incinerated.

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