RCRA Hazardous Waste Container with Gasketed Lids and Absorbent Pads

2-Gallon RCRA Hazardous Waste Container / Contenedor de 2 galones para desechos peligrosos RCRA /
Contenant à déchets dangereux RCRA de 7,6 litres (2 gallons) — Model 5002 / Modelo 5002 / Modèle 5002

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ATTENTION

*Please review these instructions carefully
before using this container.*

- For use in supervised and controlled areas.
- For solid hazardous pharmaceutical waste including P-, U- and D-listed drugs.
- Not for sharps, infectious/biohazard waste and free liquids.
- For incineration only by federally permitted hazardous waste incinerators.
- Do not overfill or force items into container.
- If container is dropped, inspect for damage before handling.
- For single use only. ⓧ Maximum Usage: 90 days.
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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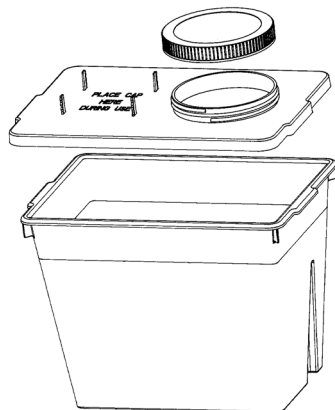
1. **To assemble**
 - a. Check that container is not damaged before placing into use.
 - b. Check that foam gasket in cover is aligned with positioning tabs.
 - c. Set lid on base, lining up handles on both. Press lid firmly to base ensuring that three snaps in front and back and two snaps on each side are firmly engaged.
2. **To mount in bracket**

Mount on wall using wall mounting bracket #435 020. Gently slide container into bracket until it clicks.
3. **To use**
 - a. With one hand, drop items into container. During use, the cap may be stored on the cap holder.
 - b. Keep container closed when not in use.
 - c. Check fill level regularly. Replace container when fill level is reached.
4. **Final closure**
 - a. Screw lid on tightly for final disposal.
 - b. To remove container from bracket, insert the bracket key, #440 020, into slots at bottom of unit, push up and release container from bracket.
5. **Disposal**
 - a. Container can be in use for maximum of 90 days.
 - b. Incineration only by a federally permitted hazardous waste incinerator.

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