WallSafe® Sharps Disposal Container

Models 125 and 150 / Modelos 125 y 150 / Modèles 125 et 150

ATTENTION

Please review these instructions carefully before using this container.

- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only. (ⓧ)
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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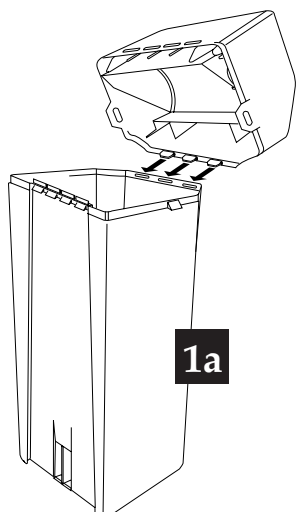
1a

1. To assemble

- Insert three tabs on cover into the three front slots in base.
- Rotate cover over base to snap tabs in slots on back and sides of container. Push down to lock cover in place.
- Check all tabs for a positive lock.

2. To mount in bracket

NOTE: This container must be wall-mounted during use.

Mount on wall using WallSafe mounting bracket #405 020. Gently slide WallSafe down into bracket until it clicks.

3. To use

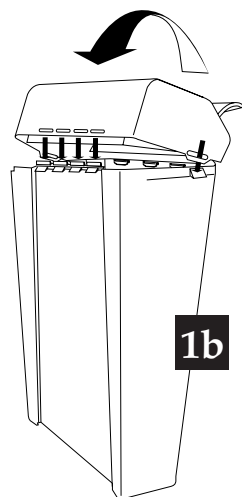
- Brush back lid with one-handed motion, dropping syringe horizontally
- Check fill level regularly. Replace container when fill level is reached.

4. Final closure

- When full, insert center tab into slot on lid for final closure.
- To remove WallSafe from bracket, insert the bracket key into slot at bottom of unit, push up and release from bracket.

5. Disposal

Dispose of container according to facility policy. WallSafes may be autoclaved or incinerated.

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