WallSafe[®] Wall Bracket and Key Set

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Use WallSafe Bracket Key to remove sharps container from bracket.

Mounting Instructions

MASONRY WALLS

1. Using the WallSafe Bracket as a template, locate it on the wall in the desired position and drill locating reference points at each screw hole.
2. Drill 1/4" diameter holes 1-1/4" deep. Blow dust out of the holes and insert plastic wall plugs, tapping them with a hammer if needed to bring them flush to the wall. Screw on the WallSafe Bracket with the provided screws.

Note: If for any reason the provided screws are not used, only pan head or round head 10-gauge screws should be substituted. In no case should countersunk screws be used.

INCLUDED IN PACKAGE

- 10 Bemis WallSafe Brackets
- 44 Plastic Wall Plugs
- 44 1-1/4" Pan Head 10-Gauge Screws

Mount WallSafe Bracket at a safely accessible height approximately 50" from top edge to floor.

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HOLLOW WALLS

Metal expanding wall fasteners that require a 3/8" drill hole size or plastic toggle expanding wall fasteners that require a 5/16" drill hole size should be used. Fasteners requiring smaller diameter holes have screw heads that are too small for secure wall bracket fastening.

Typical recommended fasteners are:

Molly fastener	CD6S	Metal fastener for 3/8" to 5/8" wallboard
Rawl anchor	2112	Metal fastener for 3/8" to 5/8" wallboard
Poly-Toggle anchor	2304	Plastic toggle fastener for 1/2" wallboard
Poly-Toggle anchor	2305	Plastic toggle fastener for 5/8" wallboard

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