3-Gallon Sharps Container with Large Opening Lid

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ATTENTION

**Please review these instructions carefully
before using this container.**

- For use in supervised and controlled areas
- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only. (X)
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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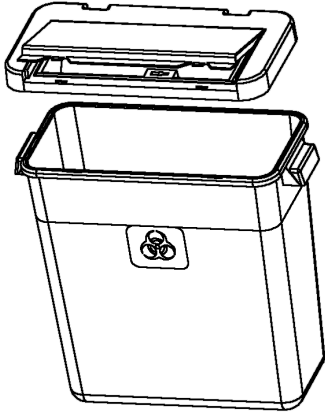
1. To assemble

- Align front of lid (lid opening) to front of base (biohazard label).
- Align tabs to holes in base. If they do not align, make sure lid and base are both facing forward.
- Press lid to base until all tabs snap into slots and are secure.

2. To mount in bracket

NOTE: This container must be wall-mounted during use.

Mount on wall using wall mounting bracket #423 000 (non-locking) or #425 000 (locking). Gently slide container down into bracket.

3. To use

- Using one hand, drop syringe horizontally into opening.
- Check fill level regularly. Replace container when fill level is reached.

4. Temporary & final closure

- Keep lid down when not in use.
- When full, insert tabs on lid into slots and press down firmly.
- Remove container from bracket.

5. Disposal

Dispose of container according to facility policy. May be autoclaved or incinerated.

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