8- and 11 Gallon Sharps and Chemotherapy Containers

Models 108, 208, 111 and 211 / Modelos 108, 208, 111 y 211 / Modèles 108, 208, 111 et 211 (30 L / 42 L)

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ATTENTION

**Please review these instructions carefully
before using this container.**

- For use in supervised and controlled areas
- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only. (X)
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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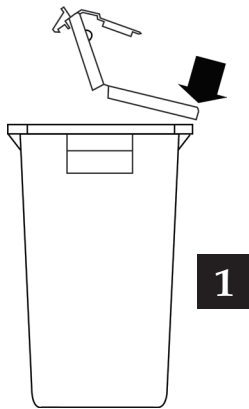
1. To assemble

- Place base on stable surface with biohazard label facing forward. Place lid on base with embossed writing toward front.
- Firmly push the back half of the lid onto the base until it snaps under the rim.
- Leave front half open to dispose of large items.

2. To use

Do not force sharps into container. Regularly check content to ensure it does not exceed the fill line.

3. For temporary closure

Gently push down on the front half of the lid, until front hooks engage rim.

Note: The round opening does not have a temporary close feature. Do not close and attempt to reopen by pulling the tab.

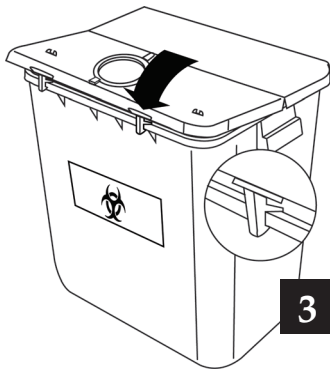
4. For final closure

- Lift both tabs at the front edge so that they clear the rim (*see A*).
- Firmly push on all the edges to ensure that the lid is snapped to the base all the way around.
- To close the small opening for final disposal, insert tab into slot and press down firmly (*see B*).

5. Disposal

Chemotherapy containers must be incinerated for final disposal.

Sharps containers may be autoclaved and/or incinerated for final disposal, according to facility policy.

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