Bemis One-Quart Phlebotomy Container

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ATTENTION

***Please review these instructions carefully
before using this container.***

- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only. (X)
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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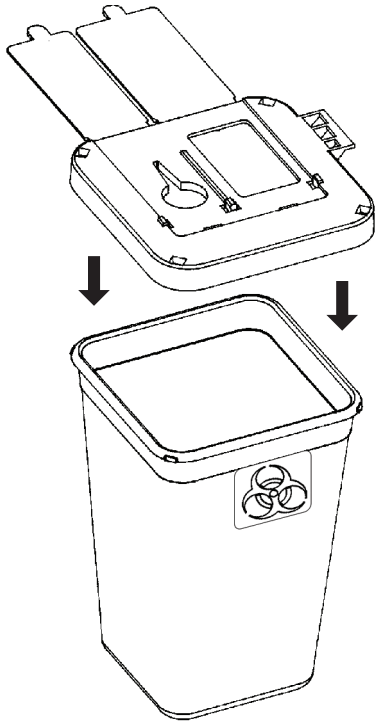
1. To assemble

Set One-Quart Phlebotomy Container base on firm surface. Set lid on base, lining up tabs with container label. Push lid firmly on base, ensuring that all four snaps are firmly engaged.

2. To mount in holder

Remove backing from adhesive mounted to bottom of Phlebotomy Holder. Firmly press on surface where Phlebotomy Container is to be mounted. Set assembled Phlebotomy container in holder.

3. To use

- Do not force sharps into container.
- Check fill level regularly. Replace container when fill level is reached.
- Keep lids closed when not in use.

4. Temporary and final closure

Temporary: With tabs of lids pointed up, snap down sides of lids into slots.

Final: Before removing Phlebotomy Container from holder, close both lids for final disposal. Firmly insert the two tabs into the slots to lock the lid closed. Press firmly to ensure tab and sides are snapped.

5. Disposal

One-Quart Phlebotomy Containers may be autoclaved or incinerated.

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