Two-Gallon Sharps Container

Model 102 / Modelo 102 / Modèle 102 (7,6 L)

*See instructions
for use on back.*

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ATTENTION

**Please review these instructions carefully
before using this container.**

- For use in supervised and controlled areas
- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only. (X)
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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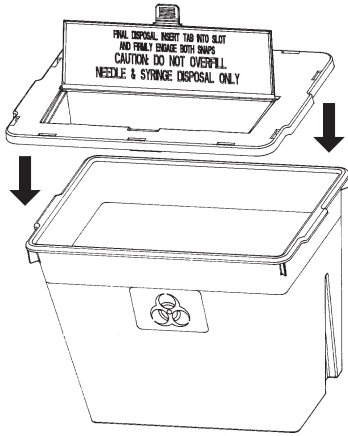
1. To assemble

- Align front of lid (lid opening) to front of base (biohazard label).
- Align tabs to slots in base. If they do not align, make sure lid and base are both facing forward.
- Press lid to base until all tabs snap into slots and are secure.

2. To mount in bracket

- Mount on wall using wall mounting bracket #445 020. Gently slide container into bracket until it clicks.

3. To use

- With one hand, drop syringe horizontally into opening.
- Check fill level regularly. Replace container when fill level is reached.

4. Temporary closure

Lid can be temporarily closed when not in use. With tab facing up, bring lid forward and press down on two front snaps. Use tab to reopen lid.

5. Final closure

- When full, bring lid forward and insert tab into slot. Press down on two front snaps to lock lid closed.
- To remove container from bracket, insert the bracket key into slots at bottom of unit, push up and release container.

6. Disposal

Dispose of container according to facility policy. May be autoclaved or incinerated.

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