# 2-Gallon Chemotherapy Container with Gasketed Lid and Screw-On Cap

Model 202 / Modelo 202 / Modèle 202 (7,6 L)

## ATTENTION

**Please review these instructions carefully before using this container.**

- For use in supervised and controlled areas
- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only. (X)
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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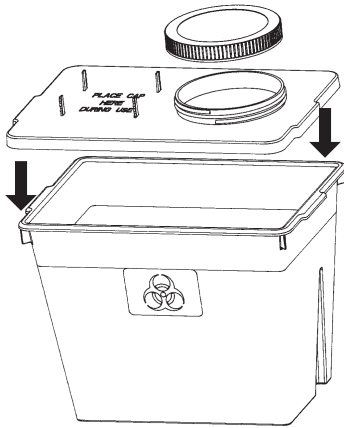
### 1. To assemble

- Set lid on base, lining up handles on both.
- Press lid firmly to base ensuring that three snaps in front and back, and two snaps on each side are firmly engaged.
- The parts will not snap together if not correctly aligned. Be sure handles on both lid and base meet.

### 2. To mount in bracket

- Mount on wall using wall mounting bracket #445 020. Gently slide container into bracket until it clicks.

### 3. To use

- With one hand, drop sharps into container. During use, the cap may be stored on the cap holder.
- Keep container closed when not in use.
- Check fill level regularly. Replace container when fill level is reached.

### 4. Final closure

- Screw lid on tightly for final disposal.
- To remove container from bracket, insert the bracket key into slots at bottom of unit, push up and release container from bracket.

### 5. Disposal

Chemotherapy container must be incinerated for final disposal.

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