5-Quart Sharps Container

Model 175 / Modelo 175 / Modèle 175 (4,8 L)

*See instructions
for use on back.*

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ATTENTION

***Please review these instructions carefully
before using this container.***

- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only. (X)
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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1. To assemble

- Align front of lid (lid opening) to front of base (biohazard label).
- Align tabs to slots in base. If they do not align, make sure lid and base are both facing forward.
- Press lid to base until all tabs snap into slots and are secure.

2. To mount in bracket

- Mount on wall using wall mounting bracket #445 020. Gently slide container down into bracket until it clicks.
- Can also be used with #1051 locking wall cabinet.

3. To use

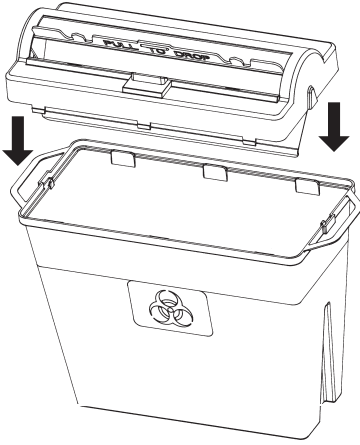
- Place sharps horizontally into cradle opening. Pull tab on cylinder forward if necessary to drop sharps into container.
- Check fill level regularly. Replace container when fill level is reached.

4. Final closure

- When full, insert tabs on cylinder into slots for final closure.
- To remove container from bracket, insert the bracket key into slots at bottom of unit, push up and release container from bracket.

5. Disposal

Dispose of container according to facility policy. May be autoclaved or incinerated.

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