# 5-Quart Sharps Container

Model 175 / Modelo 175 / Modèle 175 (4,8 L)

*See instructions  
for use on back.*

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## **ATTENTION**

***Please review these instructions carefully  
before using this container.***

- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only. (X)
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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**1. To assemble**

- Align front of lid (lid opening) to front of base (biohazard label).
- Align tabs to slots in base. If they do not align, make sure lid and base are both facing forward.
- Press lid to base until all tabs snap into slots and are secure.

**2. To mount in bracket**

- Mount on wall using wall mounting bracket #445 020. Gently slide container down into bracket until it clicks.
- Can also be used with #1051 locking wall cabinet.

**3. To use**

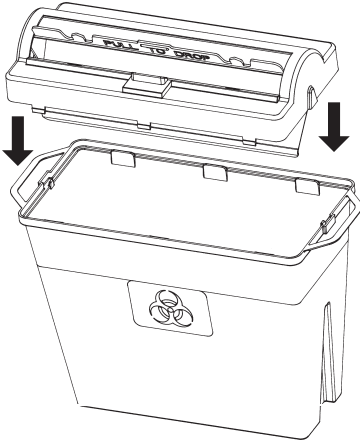
- Place sharps horizontally into cradle opening. Pull tab on cylinder forward if necessary to drop sharps into container.
- Check fill level regularly. Replace container when fill level is reached.

**4. Final closure**

- When full, insert tabs on cylinder into slots for final closure.
- To remove container from bracket, insert the bracket key into slots at bottom of unit, push up and release container from bracket.

**5. Disposal**

Dispose of container according to facility policy. May be autoclaved or incinerated.

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